PROT-CONTROL

Presentación / Presentation:
4 x 1 mL. Ref.: 42220
Conservar entre /Store: 2 - 8 °C.

**SUERO CONTROL
IMMUNOPROTEINAS**

Quantitative determination of plasma protein.

Only for *in vitro* use in clinical laboratory (IVD)

CHARACTERISTICS OF PRODUCT

The PROT CONTROL is a liquid serum from human origin containing protein concentrations suitable to evaluate the accuracy and precision of plasma protein determinations by immunoturbidimetry method.

REAGENTS COMPOSITION

PROT-CONTROL

Control. Human serum. Sodium azide 0.95 g/L. Protein concentrations are stated on the list below.

PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

REAGENT PREPARATION AND STABILITY

The calibrator is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

Opened vials, stable for 30 days at 2-8°C .

CALIBRATION

The concentration values of the Control are traceable to Standard Reference Material CRM 470/RPHHA (Institute for Reference Materials and Measurements, IRMM) (all constituents except antithrombin III), and the 2nd International Standard for antithrombin III, human 96/520(1997) WHO (for antithrombin III).

PROCEDURE

To be used in immunoturbidimetric assays.

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CHEMELEX, S.A.